



## Blood Spill Guidelines

Please note that Det-Sol 5000<sup>®</sup> powder is hygroscopic, meaning that it draws in moisture and disinfects upon contact. As a precautionary measure, we suggest the area also be washed down with solution.

Det-Sol 5000<sup>®</sup> has been included in major Australian branded First Aid and Blood spill kits. Please take note of the following safety guidelines.

- Use protective gloves and mask
- Sprinkle one or more sachets of Det-Sol 5000<sup>®</sup> powder over affected area, depending on size of spill
- Leave for 30 seconds to 10 minutes depending upon volume of contaminant
- Cover with paper towel
- Remove paper towel and congealed mass disposing into double rubbish bag or bio-hazard bag
- Add and dissolve one sachet of Det-Sol 5000<sup>®</sup> in one litre of water and wash down affected area with solution
- Dispose of all waste thoughtfully